

# SERVICE BULLETIN

**Product: SL-14**

## Why do the changes?

During the recent years of selling the hand held slit lamp SL-14, we noted that the operators sometimes press the battery release tab while the bulb is illuminated. This cause spark on the battery contacts and burned the contact plates resulting to no illumination light.

## What changes needed?

As shown on Fig.1 and Fig.2 , the electrical contact between the battery and slit lamp is made by 3 spring loaded contact pins on the slit lamp side that are pressed to the contact plates on the battery.

This contacts will be replaced with a 5 pin board connector and socket (Fig.3 and Fig.4). This change will be made on both the Slit Lamp and the Charger base



## How to differentiate between the two version?

To identify the new version from the old, the part number of all 3 items will be change slightly

Description	Old Part No.	New Part No	Starting Serial no
Slit lamp	ASL14	ASL14K	210001
Charger			
A-Type	ASL-CB14	ASL-CB14K	240001
C-Type	ASL-CB14BK	ASL-CB14BK	
Battery	ASL-PP14	ASL-PP14D	

## Compatibility

Although just little changes have been made, the old and new version are not compatible at all. This means that if a customer has an SL-14 and wants to purchase a new battery, it is essential to know if it is the new or old version they got

- If it is the new version, there is no problem supplying these parts
- If it is the old version, we recommend to have the complete SL-14 upgraded to new version